

STIC IntelConnect iOS App Instructions

STIC IntelConnect = online access to your patient's images and reports




Downloading and setting up the STIC IntelConnect iOS App is super easy

Not yet registered for STIC IntelConnect?

No problem

Click the link at the end of these instructions and complete the registration form


1. Use your iPad or iPhone and head to the App Store. Type "Intelerad" into the App Store search function and Install the App

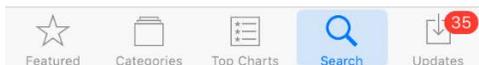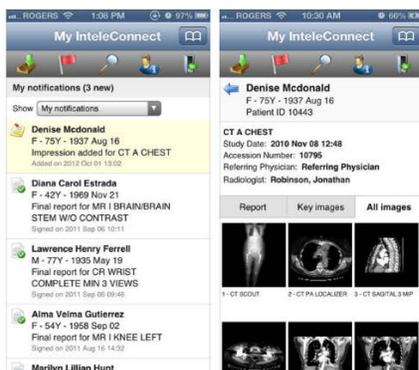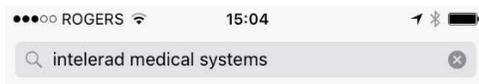


2. Open the IntelConnect App. You will need to register yourself once before using the App. If you see this warning message, click “Save”



Then you will see this window.

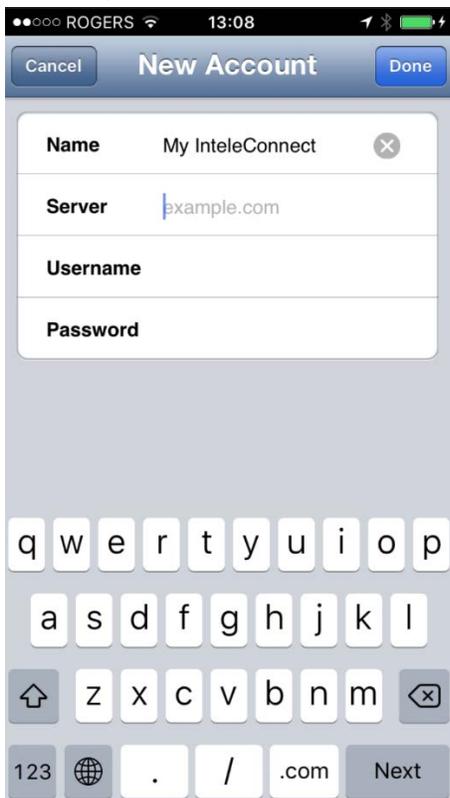

3. Enter this server name <https://radiology.stittsvilleimaging.ca/Portal/app/>

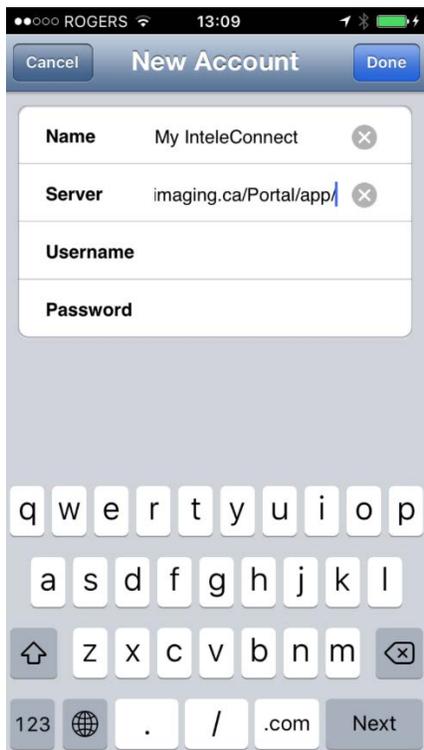

4. Enter the username and password you received from us and press “Done”. In this example, I have inserted my own username and password.



5. You will now see this window which is the Login window you will see from now on when you open the App. Enter your Login credentials once more.

My IntelConnect

Stittsville Imaging Centre Inc

Username

Password

Log In

IntelConnect
IntelPACS-4-12-1-R16

powered by
Intelerad
Distributed Radiology Solutions™

© 2001-2016 Intelerad Medical Systems Incorporated - All rights reserved

My IntelConnect

Stittsville Imaging Centre Inc

Username
gdavies

Password
●●●●●●●●●●

Log In

IntelConnect
IntelPACS-4-12-1-R16

powered by
Intelerad
Distributed Radiology Solutions™

© 2001-2016 Intelerad Medical Systems Incorporated - All rights reserved

6. You are now all set to review your patient's images and reports

7. Remember, when you Login you won't see anything until you have referred patients to us

Learn More and Find Our Requisition at
www.stittsvilleimaging.ca/health-care-providers/